**TIGER**  
D E N T A L

Innovations in orthodontics

Patienteninformation  
Patient information

## Flex Developer 2.0

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### Dear patient!

The Flex Developer (FD) is tried and tested and is extremely reliable and stable. Long term tests have also shown that the spring rods do not lose their strength after months of use in the mouth.

### What might damage my treatment device?

Parts of the treatment device can loosen if the mouth is opened extremely wide, such as when yawning, biting off large chunks of food (apple, Big Mac, etc.) or with intense kissing. Spices such as curry, paprika or pumpkin seed oil do not cause any damage but can cause discolouration of the spring rods.

### How can you help yourself in an emergency?

- If you have pressure sores, you can place a piece of paper (e.g. kitchen roll, newspaper) between the spring rod and the mucous membranes. The gums will calm down and heal up.
- If something should get stuck and make it difficult to open your mouth, or if you have any other difficulties (e.g. claustrophobia, acute pressure sore) you can deactivate the FD yourself temporarily. To do that, you can bend up the hook closure a little with small pliers (e.g. nail clipper, wire cutter, electric pliers) (Fig. 1–2). Please clean the pliers with alcohol or disinfectant before using them in your mouth. Then push your lower jaw forward and the FD will unhook. After that, the FD can be remounted in the main arch in the upper jaw (Fig. 3). If nothing else, the spring rod can also be separated and completely removed from the mouth.

With reference to the above scenario, please make sure that you only use material which is clean and will not cause any infection in the mouth. You should also be extremely cautious in case you injure yourself. If in doubt, please contact your doctor.

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